

Crane Breast Milk Storage Bag | USER GUIDE

This storage bag has been manufactured to ensure safety and reliability for what means the world to you, your infant.

! WARNING: READ AND SAVE THESE INSTRUCTIONS
Failure to follow all the instructions listed below may result in bacteria growth, and or serious infection.

CAUTION:

- Avoid over filling the breast milk storage bag, fill to max line only.
- Breast milk expires three days in refrigerator and 3 months in the freezer.
- DO NOT microwave, boil, or steam Breast Milk Storage Bag
- Use WARM running water only to thaw Milk Storage Bag
- Milk Storage Bags are not suitable to be reused. DO NOT freeze breast milk again after thawing. Keep out of reach of children.

ATTENTION:

- Breastmilk will expand when frozen. Be sure NOT to overfill the bag.
- If bag is overfilled, milk can expand and cause bag to tear, causing loss of milk
- Crane Milk Storage Bags can only be used with Crane Breast Pumps (EE-9002, EE-9003, EE-9004)
- When transferring milk to storage bag use the Crane Milk Collection Kit
- Storage bags are single use only, discard of bag after it has been emptied.
- Always make sure parts have been clean before use.
- Pumping directly into bag requires two hands for safe milk transferring.

Failure to observe or follow these instructions and other warnings can negatively impact the performance of the product

INCLUDED: 30 6oz bags and 1 adapter

PARTS:



MAINTENANCE:

Prior to each use sanitize adaptor and bottle parts. After each use, wash with warm soapy water and allow to air dry

INSTRUCTIONS:

1. Always clean hands prior to use. Make sure to wash and dry hands with soap and water before handling expressed milk
2. Remove cap off the bag to open the storage bag. You will hear the safety seal of the plastic break. Place cap to the side for later use. **KEEP CAP OUT OF REACH OF CHILDREN AT ALL TIMES!**
3. Attach Milk Bag adapter to the Crane Breast Shield Body in the Milk Collection Kit
4. Attach your Milk Storage Bag to the adapter
5. Begin following your Crane Breast Pump instructions to fill the storage bag
6. When you have reached the Max level of Milk Storage Bag carefully remove the bag from the adapter and screw cap firmly onto Milk Storage Bag. **NOTE: Leaving a small amount of air in bag.**
7. Label bag and cap with date using permanent for easy viewing
8. Store and freeze Milk Storage Bag flat.
9. When ready to use milk, remove from freezer to thaw or use warm running water. **DO NOT MICROWAVE, STEAM OR BOIL BAG**



FEEDING:

1. After Milk Storage Bag is thawed locate your Crane Bottle Kit.
2. Remove protective bottle cap and unscrew bottle nipple from bottle.
3. Attach storage adapter to bottle nipple ring.
4. Remove the cap from Milk Storage Bag.
5. Carefully squeeze air out of Milk Storage Bag
6. Secure Storage Bag on to Storage adapter.
7. You are now ready to feed your baby!
8. You may gently squeeze the bag during feeding to increase the flow.

WARRANTY:

This product is warranted to be free from manufacturing defects under normal use and conditions for **1 year**. This warranty **requires a copy of your dated proof of purchase**.

WARRANTY IS VOID IF THE DEFECT IS DUE TO: Accidental damage, misuse, abuse, improper repair, or failure to follow instructions provided.